

# OSSIX<sup>®</sup> Plus

TIPS FOR FIRST TIME USERS

Ossifying Collagen Barrier Membrane


Maximizing OSSIX<sup>®</sup> Plus

.....  
Thank you for choosing OSSIX<sup>®</sup> Plus,  
the sugar cross-linked resorbable collagen membrane

## Please carefully read the below guidelines prior to your first use of OSSIX® Plus.

- 1 It is recommended to start with simple cases and get familiar with the material's behaviour and handling features.
- 2 Trim OSSIX® Plus with curved scissors and do not leave cuts in the membrane. It is recommended to trim the membrane prior to hydration.
- 3 Hydrate OSSIX® Plus with sterile saline until fully hydrated.
- 4 **Important:** do not try to suture through the membrane – it may tear.
- 5 To stabilize the membrane, suture over it with periosteal sutures.
- 6 If prematurely exposed, re-suture if flaps are mobile; if the membrane is stable, no need to re-suture.
- 7 Instruct your patient to rinse with chlorhexidine and to avoid hard or hot food. OSSIX® Plus will last up to 5 weeks if exposed.
- 8 When re-entering the site, you might see membrane remnants. Do not remove them as these will ossify over time.
- 9 Please refer to the OSSIX® Plus IFU for full product information, indications and contraindications. [LINK](#)

### OSSIX® Plus Sizes

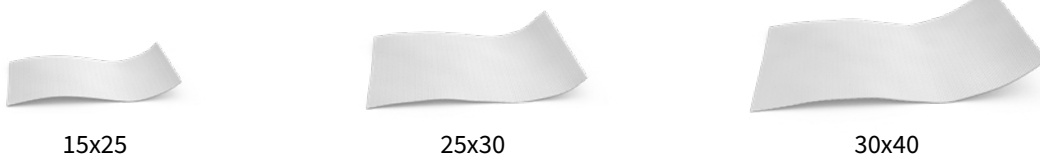

### WHY OSSIX®?

OSSIX® safe and effective regenerative solutions are set to change the way you work and the way your patients feel. Our safe and proven OSSIX® solutions for GBR and GTR significantly reduce the need for repeated procedures, simplify complex procedures, and are cost-effective. Your patients will benefit from enhanced healing, being able to quickly return to their routine – both contributing to a higher compliance rate.

Leading dental surgeons frequently use OSSIX® Regenerative Solutions and vouch for their simplicity and efficacy. Trusted over more than 20 years, OSSIX® is the go-to solution for a wide variety of procedures.

### MORE TIPS FOR YOU

