



G&H® Wire Company

2165 Earlywood Drive
Franklin, Indiana 46131 USA

(317) 346-6655 International

(317) 346-6663 Fax

E-mail: ghmail@ghwire.com

G&H Mirror Cleaning and Sterilization Recommendations

1. Gently wash the mirror with mild hand soap using alcohol to remove greasy substances such as lipstick.
2. Dry the mirror with a soft cloth lightly and then air dry until all moisture has evaporated.
3. Any spots left on the mirror will not come off after heat or autoclave sterilization. If the mirror cannot be cleaned right away, keep it wet or covered with a wet towel.
4. Never place a mirror into an ultrasonic unit.
5. Keep the mirror out of contact with all other instruments. Wrap the mirror into a microfiber cloth or paper towel and place it in the sterilizing pouch. The wrapping will absorb any steam that builds up on the mirror inside the pouch.
6. Follow the sterilization recommendations as provided by the manufacturer of your sterilizer.