

# INSTRUCTIONS FOR USE OF MICRO SAW BLADES

## **Description:**

Surgical stainless steel Micro Saw Blades are available in various diameters and lengths. They are available for the Micro Oscillating Saw, Micro Sagittal Saw, Micro Osseoscalpel Saw and Micro Reciprocating Saw.

## **Application:**

For use in conjunction with our Micro Saw Handpieces. To be used solely for the sawing and trimming of bone in Oral, Maxillofacial, Craniofacial, Hand and Plastic Surgery.

Surgical sawing as necessitated by the patient's condition and treatment provided by the consultant.

## **Intended User:**

Qualified clinicians with the required specialist knowledge.

## **Patient Population:**

No restriction on patient age or population.

## **Packing & Sterilisation:**

Micro Saw Blades are supplied sterile for single use only. Packed individually in a box, sterilised by Gamma Irradiation in accordance with EN 11137.

## **Precautions:**

- Must only be used for its intended purpose.
- This device is supplied sterile, intended for single use only.
- Not to be re-sterilised once open.
- Re-use of single use devices present a risk of infection to the patient.
- Do not use if the packaging is damaged or opened. Dispose of immediately if packaging is found to be breached in any way.
- The material is Latex Free.

## **Storage:**

- To be stored under normal ambient conditions.
- Keep away from direct sunlight and must be kept dry.
- Life of the product is limited to the expiry date on the label.

## **Disposal:**

Can be disposed of without any special instructions and poses no danger if standard medical waste disposal regulations are observed.



Manufacturers name



Manufacturing date

LOT

Lot number



Expiry date

STERILE R

Method of sterilisation



Indicates single use



Refer to instructions



Keep dry



Keep out of direct sunlight



Do not use if packaging is damaged



Consult instructions for use

CE 1639

**MERCIAN**

Surgical Instruments of Excellence

Mercian Surgical Supply Co. Ltd.  
10 Topaz Way, Topaz Business Park,  
Bromsgrove, Worcestershire B61 0GD.

Tel: +44 (0) 844 879 1133

Email: [info@merciansurgical.com](mailto:info@merciansurgical.com)

Date of issue:

**December 2019**

Revision number:

**3**