

## MAXIMUM PROTECTION

Cushions absorb up to 85% of impact forces.\* The 1" (3 cm) EVA foam creates a low profile that is easy for patients to step on and off. The non-skid bottom helps prevent slipping when stepping onto the Cushion. The vinyl cover of the floor cushion meets CA #117, Section E Flame Retardancy Standard.

Cushions with beveled edges easily accommodate wheelchairs, walkers, and table trays. The Two-Tone Cushion is easier to see in low light conditions, or for a sight impaired patient.

## USE INSTRUCTIONS

1. Undo the hook-and-loop closure and unfold the Floor Cushion.
2. Make sure the "This Side Down" label is facing the floor.
3. Position the Cushion lengthwise, with the straight (non-beveled) edge flush to the side of the resident's bed.

**⚠ WARNING** Inform resident and staff to take care when stepping on or off of the Cushion to avoid tripping. Follow your facility's procedures for cleaning spills in patient care areas. DO NOT place Floor Cushions on or around damp or wet areas. Clean spills on, around or under Floor Cushions. Liquids under Floor Cushions may cause a slip and fall accident.

**⚠ CAUTION** When used with a Low Bed, be sure to leave enough room between the bed and the Cushion. If the Cushion is too close when the handrail is rotated, it could tear the Cushion cover.

**⚠ CAUTION** Posey Glow strips will fade after two hours unless recharged by light exposure for approximately twenty minutes.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture of high humidity that may damage product materials.

When not in use, fold the Cushion into thirds for easy, compact storage. When folded, the Cushion is only 3" (8cm) thick. Use the hook-and-loop closure as a carrying handle. The Cushion may also be stored in a hanging position using the yellow loop.

**⚠ CAUTION** DO NOT store the Cushion on the beveled edge as this may cause the edge to roll up.

NOTE: Due to the random possibility of falls, the Posey Company makes no guarantee, express or implied, that the user is protected from hip trauma.


## CLEANING INSTRUCTIONS

The vinyl cover easily wipes clean with a liquid disinfectant and a soft cloth.

**⚠ CAUTION** DO NOT use any disinfectant or cleaning products that contain Phenol.



Simple precautions can help create a safer environment. The Posey Floor Cushions help to minimize injuries from a fall.

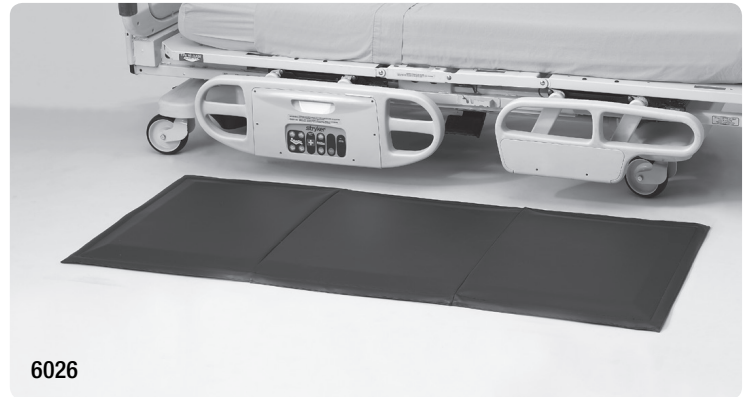

## POSEY FLOOR CUSHIONS

<b>REF</b>	<b>6020</b>	Standard Floor Cushion: 67"L x 26"W x 1"H (170 cm x 66 cm x 3 cm)
<b>REF</b>	<b>6021</b>	Two-Tone Floor Cushion: 67"L x 26"W x 1"H (170 cm x 66 cm x 3 cm)
<b>REF</b>	<b>6023</b>	Beveled Floor Cushion: 70"L x 38"W x 1"H (178 cm x 97 cm x 3 cm)
<b>REF</b>	<b>6025</b>	Beveled Floor Cushion, Wide: 70"L x 44"W x 1"H (178 cm x 112 cm x 3 cm)
<b>REF</b>	<b>6026</b>	Beveled Floor Cushion, Narrow: 70"L x 29"W x 1"H (178 cm x 74 cm x 3 cm)
<b>REF</b>	<b>6027</b>	Beveled Floor Cushion, Premium: 70"L x 38"W x 1"H (178 cm x 97 cm x 3 cm)

## DISPOSAL

Clean with liquid disinfectant to remove contamination before disposal.

**⚠ WARNING** Dispose of the product per the facility's policies.

\* All models tested according to ASTM protocols for shock absorbency.