

DESCRIPTION OF PRODUCT: Elbow or Knee Sleeve



8168M

ADULT SecureSleeve® Protector

SIZE	LENGTH	LIMB CIRCUMFERENCE
8168XS	10" (25 cm)	3"-12" (8-30 cm)
8168S	11" (28 cm)	3"-14" (8-36 cm)
8168M	13" (33 cm)	6"-17" (15-43 cm)
8168L	14" (36 cm)	8"-21" (20-53 cm)
8169	18" (46 cm)	14"-27" (36-69 cm)

INFANT/PEDIATRIC SecureSleeve® Protector

SIZE	LENGTH	LIMB CIRCUMFERENCE
8168I	3" (8cm)	1 ½"-4" (4-10 cm)
8168IL	4½" (11 cm)	1 ½"-4" (4-10 cm)
8168P	7" (18 cm)	3"-10" (8-26 cm)

8162 Attachment Clips, 1 dozen

Indications for Use

- Patients assessed requiring an external limb split to immobilize a limb or extremity.
- Follow your hospital's restraint policies and procedures which are in compliance with CMS guidelines and state laws.

Contraindications

- Do not use this device with someone who has continued highly aggressive or combative behavior, self-destructive behavior, or who is deemed to be an immediate risk to themselves or others.

Precautions

- Avoid using on a patient with a dislocation or fracture on the limb, or if an IV or wound site could be compromised by the device.
- Check the patient regularly to ensure that circulation is not impaired. Injury may occur if the sleeve restricts circulation when applied.
- Never alter or repair this product. Always inspect before each use: Check for broken stitches or parts; torn, cut or frayed material; or hook-and-loop fasteners that do not hold securely.
- Do not use soiled or damaged products. Doing so may result in serious injury or death. Dispose of damaged products per facility policy for biohazardous material.
- Do not allow patients to ingest product material.
- **Warning:** Additional or different body or limb restraints may be needed to reduce the risk of the patient getting access to the line, wound, or tube site. (See Posey® Catalog)

Application Instructions


1. Open the SecureSleeve® Protector by releasing the hook-and-loop adjustment straps.
2. Orient the sleeve so that the plastic buckles are positioned towards the patient. Infant sized sleeves will orient the strap towards the patient.
3. Adjust the sleeve to the desired position on the limb. The sleeve should be centered over the elbow with the opening towards the inside of the arm. For leg application, the sleeve should be centered over the knee with the opening towards the front of the leg.
4. Secure the sleeve by threading the hook-and-loop strap through the plastic buckle and securing back onto itself. To secure an infant sized sleeve, wrap the hook-and-loop strap around the limb and attach to the corresponding hook-and-loop strip. Leave enough room to easily insert two fingers between the device and the patient's limb to maintain adequate circulation.

Optional:

5. To reduce the sleeve size (i.e. for a limb that is in between sizes), fold the outside panel with hook-and-loop towards the outer lining, exposing the white inner lining, minimizing the sleeve width. Continue by following steps 2-4 as described above.

Clip Application

The optional clips connect the SecureSleeve to the patient's sleeve and help prevent sliding off the patient's arm.

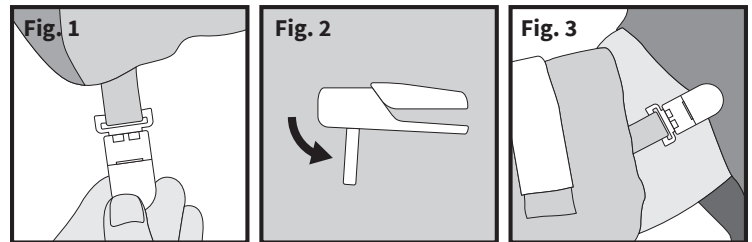

- To apply clip to SecureSleeves, insert clip through sleeve loop (Fig. 1)
- To open clip, pull tab and apply to patient's sleeve (Figs. 2 and 3)

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Laundering Instructions

