
INSTRUCTIONS

EN
English

1. Select the area for the placement of the Grip-Lok. Note: catheter, tube or line should be already placed/inserted.
2. Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
3. Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.
4. Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.
5. Remove the interior liner to expose the adhesive areas.
6. Place the catheter, tube or line in the center of the exposed adhesive.
7. Secure top flap over the catheter, tube or line and apply gentle pressure to the center fabric area over the adhesive.

Direction: for added skin protection, use skin prep pad prior to application of stabilization device.
Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
Note: Use of an alcohol swab may help removal of securement device from the skin.
Note: re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
Note: Hypoallergenic.

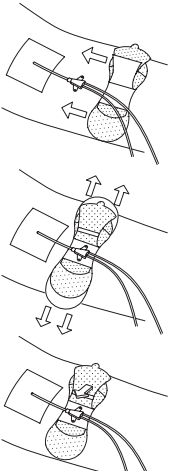2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA

Phone: +1 920 751 4300
tidiproducts.com/securement

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Grip-Lok is a registered trademark of TIDI Products, LLC.

Universal Securement Device

REF	Description
3300MWA	Medium Universal Securement with Wide silicone adhesive area

3300MW	Medium Universal Securement with Wide silicone adhesive area
--------	--

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