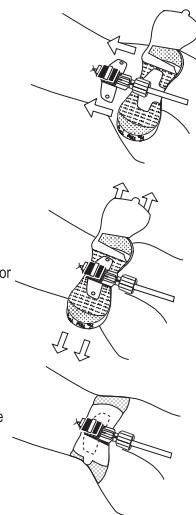
INSTRUCTIONS

EN
English

- Select the area for the placement of the Grip-Lok. Note: The arterial cannula should be already inserted in the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
- Open the top flap; remove the interior liner and hold the flap open.
- Slide the Grip-Lok under the arterial cannula and adhere in the exposed adhesive cut-out space provided.
- Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Secure the top flap over the arterial cannula with the FloSwitch portion coming through the center cut-out.

Direction: for added skin protection, skin prep may be used prior to application of securement device.
Note: replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
Note: use of an alcohol swab may help removal of securement device from the skin.
Note: re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
Note: Hypoallergenic

2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
Phone: +1 920 751 4300
tidiproducts.com/securement

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Grip-Lok®

BD Arterial Cannula with FlowSwitch™ Securement Device

REF

Description

3301MSC-BD Securement Device for the BD Arterial Cannula with FloSwitch™

STERILE R

MD

BD and **FloSwitch** trademarks
are property of Becton,
Dickinson and Company.
Grip-Lok is a registered
trademark of TIDI Products, LLC.

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