
INSTRUCTIONS **EN** English

1. Compare PICC/CVC wing shape to foam cavity in the Grip-Lok® to confirm compatibility.
 2. Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
 3. Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
 4. Open the top flap, remove the interior liner and hold the flap open.
 5. Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
 6. Secure the top flap over the catheter.
 7. Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Direction:** for added skin protection, skin prep may be used prior to application of securement device.
Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
Note: Use of an alcohol swab may help removal of securement device from the skin.
Note: Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
Note: Hypoallergenic.

2020-11-01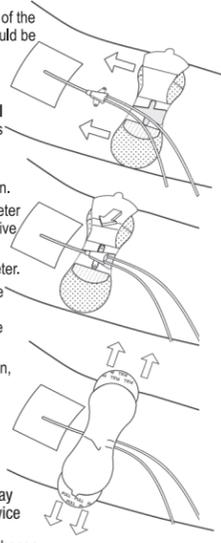

TIDI® Products, LLC
 570 Enterprise Drive
 Neenah, WI 54956 USA
 Phone: +1 920 751 4300
 tidiproducts.com/securement

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Grip-Lok is a registered trademark of TIDI Products, LLC.

Universal Securement Device

REF	Description
3304MCS-BA	Catheter Securement for Bard® PICC and CVC
3306MCS-NA	Grip-Lok® Catheter Securement for Navilyst Medical Xcela® PICC
3308MCS-MC	Grip-Lok® Securement for Medcomp® Dialysis Catheter

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