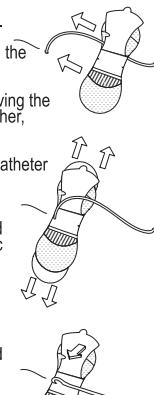
INSTRUCTIONS**EN**

English

- Select the area for the placement of the Grip-Lok.
Note: The catheter should be already inserted in the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
- Open the top flap and hold the flap open.
- Position the Grip-Lok with the catheter in the groove at edge of the foam section.
- Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Remove the interior liner and place the catheter over the foam and onto the center of the exposed adhesive surface.
- Secure the top flap over the catheter and apply gentle pressure to the center fabric area over the catheter adhesive.

Direction: for added skin protection, skin prep may be used prior to application of securing device.

Note: Replace securing device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: Use of an alcohol swab may help removal of securing device from the skin.

Note: re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

Note: Hypoallergenic.

2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA

Phone: +1 920 751 4300
tidiproducts.com/securement

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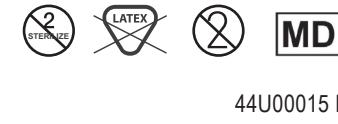
Grip-Lok®

Medium Epidural Catheter Securement Device

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