REF 3309MCS-TA-2

- (EN) Catheter Fixation for ARROW® PICC/CVC  
 (DE) Katheter Fixierung für ARROW® PICC/CVC  
 (FR) Fixation de Cathéter pour ARROW® PICC/CVC  
 (IT) Fissaggio del catetere ARROW® PICC/CVC  
 (NL) Catheter Fixatie voor ARROW® PICC/CVC  
 (SV) Kateter Fixering för ARROW® PICC/CVC  
 (ES) La Fijación del Catéter para el ARROW® PICC/CVC  
 (PT) Fixação de catéter para ARROW® PICC/CVC  
 (EL) Η δέσμευση για ARROW® PICC/CVC  
 (FI) Katetri Pakkomielle ARROW® PICC/CVC  
 (DA) Kateter Fiksering for ARROW® PICC/CVC  
 (IS) Catheter Fixation pentru ARROW® PICC/CVC  
 (NO) Kateter Fiksring for ARROW® PICC/CVC  
 (PL) Catheter Utrwalania dla ARROW® PICC/CVC

44U00016 Rev.0.0

TIDI® Products, LLC  
 570 Enterprise Drive  
 Neenah, WI 54956 USA  
*Grip-Lok is a registered trademark of TIDI Products, LLC.*

Emea distributor :  
 TeleflexMedical Europe  
 IDA Business and Technology Park,  
 Dublin Road, Athlone, Co. Westmeath, Ireland  
 Phone + 353 (0)9 06 46 08 00  
[www.teleflexmedical.com](http://www.teleflexmedical.com)

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## INSTRUCTIONS EN English

- Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
  - Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
  - Open the top flap, remove the interior liner and hold the flap open.
  - Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
  - Secure the top flap over the catheter.
  - Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Direction:** for added skin protection, skin prep may be used prior to application of securement device.  
**Note:** replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.  
**Note:** use of an alcohol swab may help removal of securement device from the skin.

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