
## INSTRUCTIONS

EN

English

- Select the area for the placement of the Grip-Lok. Note: The feeding tube should be already inserted in the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Remove the bottom liner by holding fabric section and pulling-off liner. Do not remove the top liner at this time.
- Place the Grip-Lok on the site by continuing to hold by the fabric and press onto the skin with gentle pressure. The top liner should now be removed.
- Position the feeding tube in the center of the top exposed adhesive strip between the fabric and the white hook section.
- Secure the feeding tube by folding over the fabric section until it meets with the white hook portion and apply gentle pressure to the back of the entire fabric area.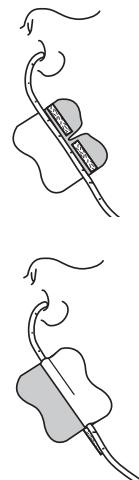

Note: to remove or adjust secured tube, hold hydrocolloid adhesive down to skin while opening top fabric section.

Note: the Grip-Lok 2100NGH may be cut in half prior to removal of liners to reduce the size.

Note: Replace security device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: Use of an alcohol swab may help removal of security device from the skin.

Note: Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

Note: Hypoallergenic.

2020-11-01

TIDI® Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA

Phone: +1 920 751 4300  
tidiproducts.com/securement

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Grip-Lok®

## Nasogastric Hydrocolloid Securement Device

REF	Description
-----	-------------

2100NGH Hydrocolloid Securement. For lines and tubes 1-2 mm in diameter. Small Nasal Gastric Size.

44U00017 Rev. 00

Grip-Lok is a registered trademark of TIDI Products, LLC.

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