Grip-Lok®

Universal Securement Device

REF

Description

GL3352 Medium securement device with wide adhesive area

STERILE R

44U00019 Rev 00

Distributed by:
Wolf Medical Supply, Inc.
 Sunrise, FL 33325
 800.335.9653
www.wolfmed.com


Grip-Lok is a registered trademark of TIDI Products, LLC.
 Wolf-Pak is a registered trademark of Wolf Medical Supply, Inc.

INSTRUCTIONS

EN

English

- Select the area for the placement of the Grip-Lok. Note: catheter, tube or line should be already placed/inserted.
 - Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
 - Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.
 - Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.
 - Remove the interior liner to expose the adhesive areas.
 - Place the catheter, tube or line in the center of the exposed adhesive.
 - Secure top flap over the catheter, tube or line and apply gentle pressure to the center fabric area over the adhesive.
- Direction:** for added skin protection, use skin prep pad prior to application of stabilization device.
- Note:** replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
- Note:** use of an alcohol swab may help ease removal of securement device from the skin.
- Note:** Hypoallergenic

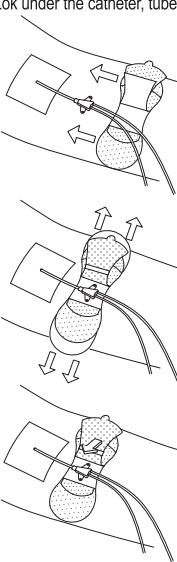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