INSTRUCTIONS

EN English

- Select the area for the placement of the Grip-Lok.
- Note: The tube should be already inserted or positioned on the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Remove half the bottom liner from the Grip-Lok then remove the top liner.
- Position the tube in the center of the top exposed adhesive strip between the fabric and the white hook section and secure Grip-Lok to skin.
- Secure the feeding tube by folding over the center fabric section until it meets with the white hook portion and apply gentle pressure to the back of the fabric area.
- Secure the end fabric section OVER the distal end of the tube.
- Secure the end fabric section UNDER the proximal end of the tube.
- Remove other half the bottom liner from the Grip-and secure to skin.

Note: to remove or adjust secured tube, hold adhesive down to skin while opening top fabric section.
Note: the Grip-Lok 2100ANG may be cut in half prior to removal of liners to reduce the size.
Note: Replace security device if soiled or saturated in fluid or if device shows signs of wear or damage.
Note: Use of an alcohol swab may help removal of security device from the skin.
Note: Hypoallergenic.

2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA

Phone: +1 920 751 4300
tidiproducts.com/securement

Nasogastric Securement Device

REF	Description
2100ANG	Nasal Gastric Securement Device. For lines and tubes sized 3-5 mm in diameter

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