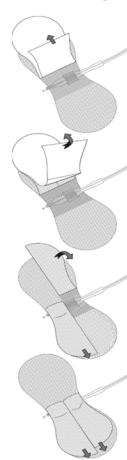
## INSTRUCTIONS

EN English

- Select the area for the placement of the Grip-Lok.
- Note: The catheter should be already inserted in the patient with Luer connection attached.
- Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open the Grip-Lok and fold back the top liner to expose center section of adhesive.
- Slide the Grip-Lok under the catheter and the Luer connector centering in the exposed adhesive area.
- Hold the Grip-Lok and catheter in position and remove the folded liner.
- Hold the Grip-Lok in position and remove one side of the bottom liner then remove the other side of the bottom liner and gently press adhesive to skin.
- Wrap top adhesive tab around the catheter pressing to conform to the shape of the catheter.
- Wrap the remaining top adhesive tab around the Luer connector pressing to conform to the shape of the connector.

**Direction:** for added skin protection, skin prep may be used prior to application of securing device.  
**Note:** Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

**Note:** Replace securing device if soiled or saturated in fluid or if device shows signs of wear or damage.  
**Note:** Use of an alcohol swab may help removal of securing device from the skin.  
**Note:** Hypoallergenic.

2020-11-01

TIDI® Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA

Phone: +1 920 751 4300  
[tidiproducts.com/securement](http://tidiproducts.com/securement)

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Grip-Lok is a registered trademark of TIDI Products, LLC.  
Patent Pending.

**Grip-Lok®**

### Peripheral IV & Arterial Catheter Securement Device

REF	Description
3300MIV	Peripheral IV & Arterial Catheter Securement
3300MART	Arterial Catheter & Peripheral IV Securement
3300MIV-KIT-2	Arterial and Peripheral IV Securement and Dressing Kit
3300MART-KIT-2	Arterial and Peripheral IV Securement and Dressing Kit

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