## INSTRUCTIONS

EN

English

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient with Luer connection attached.
2. **Prepare** the skin according to the standard hospital protocol for dressing application.
3. **Remove** one side of the release liner from the large portion of the Grip-Lok.
4. **Place** the Grip-Lok centered on the nose with the thin tab portions extending over the end of the nose.
5. **Press** the exposed adhesive down one side of the nose and onto the face.
6. **Remove** other side of the release liner from the large portion of the Grip-Lok and press the exposed adhesive down the other side of the nose and onto the face.
7. **Remove** the adhesive release liner from under one side of the thin tab portions extending over the end of the nose and wrap around the tubing.
8. **Remove** the adhesive release liner from under the other side of the thin tab portions and wrap around the tubing in the opposite direction.

**Direction:** For added skin protection, skin prep may be used prior to application of securement device.

**Note:** You can make removal of the Grip-Lok from the tubing easier by folding the end of the thin tab portions back against itself.

**Note:** Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

**Note:** Use of an alcohol swab may help removal of securement device from the skin.

**Note:** Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

**Note:** Hypoallergenic.

2020-11-01

TIDI® Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA

Phone: +1 920 751 4300  
tidiproducts.com/securement

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Grip-Lok is a registered trademark of TIDI Products, LLC.  
Patent Pending

Grip-Lok®

Large Nasogastric  
Securement Device

REF

Description

3401LNG

Large Nasal Gastric Tube  
Securement Device

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