
INSTRUCTIONS**EN**

English

- Select the area for the placement of the Grip-Lok. Note: catheter, tube or line should be already placed/inserted.
 - Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
 - Open the top flap and slide the Grip-Lok under the catheter, tube or line, centering it above the adhesive area.
 - Pull the paper backing from one side of the Grip-Lok, then the other, to secure in desired position on the skin.
 - Remove the interior liner to expose the adhesive areas.
 - Place the catheter, tube or line in the center of the exposed adhesive.
 - Secure top flap over the catheter, tube or line and apply gentle pressure to the center fabric area over the adhesive.
- Direction:** For added skin protection, use skin prep pad prior to application of stabilization device.
- Note:** Replace security device if soiled or saturated in fluid or if device shows signs of wear or damage.
- Note:** Use of an alcohol swab may help removal of security device from the skin.
- Note:** Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
- Note:** CAUTION: Intended for further processing.
- Note:** Hypoallergenic.

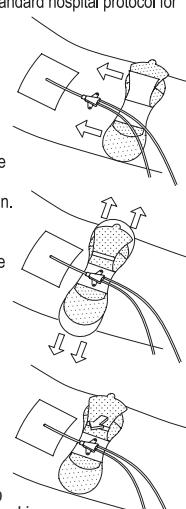2020-11-01

Grip-Lok is a registered trademark of TIDI Products, LLC.

TIDI® Products, LLC
 570 Enterprise Drive
 Neenah, WI 54956 USA

 Phone: +1 920 751 4300
tidiproducts.com/securement

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