
## INSTRUCTIONS

EN  
English

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
3. **Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
4. **Open** and separate the top fabric straps.
5. **Place** the drainage lumen between the foam strip and the back of the fabric straps.
6. **Position** the catheter so that the Y-point is centered at the keyhole in the top fabric straps.
7. **Secure** one of the fabric straps over the drainage lumen and the other over the catheter below the Y-point.
8. **Adjust** the securement of the catheter by applying the strap attachments tighter or looser as needed.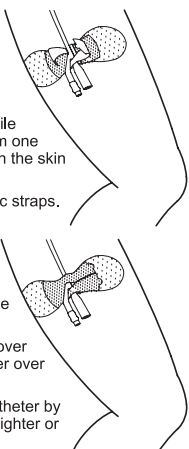

**Direction:** for added skin protection, use skin prep pad prior to application of stabilization device.  
**Note:** Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.  
**Note:** Use of an alcohol swab may help removal of securement device from the skin.  
**Note:** re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.  
**Note:** CAUTION: Intended for further processing.  
**Note:** Hypoallergenic.

2020-11-01

Grip-Lok is a registered trademark of TIDI Products, LLC.

TIDI® Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USAPhone: +1 920 751 4300  
tidiproducts.com/securementMDSS GmbH  
Schiffgraben 41  
30175 Hannover, GermanyGrip-Lok® OEM Bulk Package  
Foley Catheter  
Securement Device

REF	Description
34XXLFC-NSB	Large Foley Catheter Securement

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