INSTRUCTIONS

EN

English

1. Select the area for the placement of the Grip-Lok.
Note: The catheter should be already inserted in the patient with Luer connection attached.
2. Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
3. Open the Grip-Lok and fold back the top liner to expose center section of adhesive.
4. Slide the Grip-Lok under the catheter and the Luer connector centering in the exposed adhesive area.
5. Hold the Grip-Lok and catheter in position and remove the folded liner.
6. Hold the Grip-Lok in position and remove one side of the bottom liner then remove the other side of the bottom liner and gently press adhesive to skin.
7. Wrap top adhesive tab around the catheter pressing to conform to the shape of the catheter.
8. Wrap the remaining top adhesive tab around the Luer connector pressing to conform to the shape of the connector.

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: Use of an alcohol swab may help removal of securement device from the skin.

Note: CAUTION: Intended for further processing.

Note: Hypoallergenic.

2020-11-01

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA

Phone: +1 920 751 4300
Fax: +1 920 751 4370
tidiproducts.com/securement

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Grip-Lok is a registered trademark of TIDI Products, LLC.
Patent Pending.

Grip-Lok® OEM Bulk Package

Peripheral IV & Arterial Catheter Securement Device

REF

Description

3300MART-NSB Peripheral IV & Arterial Catheter

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