
INSTRUCTIONS **EN**
English

1. **Select** the area for the placement of the Grip-Lok. Note: catheter should be already placed/inserted.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Skin must be clean and dry. Skin prep or hair removal may be required on some patients for better adhesion.
3. **Open** the top fabric flap and remove the interior liner to expose the adhesive areas.
4. **Slide** the Grip-Lok under the catheter hub centering it in the exposed adhesive area with the distal end of the hub extending just beyond the Grip-Lok.
5. **Secure** top flap over the catheter and apply firm pressure to the fabric area over the adhesive and around and in between lumens.
6. **Pull** the paper backing from the bottom of one side of the Grip-Lok, then the other, to secure in desired position on the skin.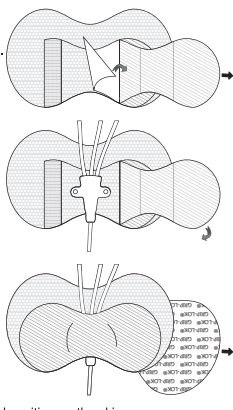

Direction: for added skin protection, skin prep may be used prior to application of securement device.
Note: Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
Note: Use of an alcohol swab may help removal of securement device from the skin.
Note: Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.
Note: CAUTION: Intended for further processing.
Note: Hypoallergenic.

2020-11-01

TIDI® Products, LLC
 570 Enterprise Drive
 Neenah, WI 54956 USA
 Grip-Lok is a registered trademark of TIDI Products, LLC.
 Phone: +1 920 751 4300
 tidiproducts.com/securement

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