# Teleflex®

Grip-Lok® OEM Bulk Package  
Arrow® CVC & PICC Securement Device

REF 3604MCS-TA-1

- (EN) Catheter Fixation for ARROW® PICC/CVC
- (DE) Katheter Fixierung für ARROW® PICC/CVC
- (FR) Fixation de Cathéter pour ARROW® PICC/CVC
- (IT) Fissaggio del catetere ARROW® PICC/CVC
- (NL) Catheter Fixatie voor ARROW® PICC/CVC
- (SV) Kateter Fixering för ARROW® PICC/CVC
- (ES) La Fijación del Catéter para el ARROW® PICC/CVC
- (PT) Fixação de catéter para ARROW® PICC/CVC
- (EL) Η δέσμευση για ARROW® PICC/CVC
- (FI) Katetri Pakkomielle ARROW® PICC/CVC
- (DA) Kateter Fiksering for ARROW® PICC/CVC
- (IS) Catheter Fixation pentru ARROW® PICC/CVC
- (NO) Kateter Fiksering for ARROW® PICC/CVC
- (PL) Catheter Utwalania dla ARROW® PICC/CVC

44U00038 Rev.00

TIDI® Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA  
Grip-Lok is a registered trademark of TIDI Products, LLC.

Emea distributor :  
TeleflexMedical Europe  
IDA Business and Technology Park,  
Dublin Road, Athlone, Co. Westmeath, Ireland  
Phone + 353 (0) 90 46 08 00  
[www.teleflexmedical.com](http://www.teleflexmedical.com)

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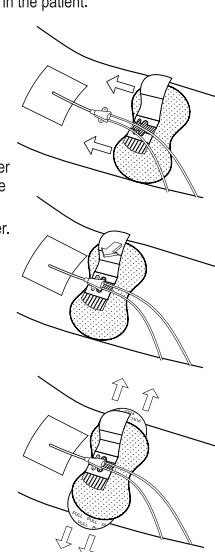
## INSTRUCTIONS EN English

1. Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
  2. Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
  3. Open the top flap, remove the interior liner and hold the flap open.
  4. Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
  5. Secure the top flap over the catheter.
  6. Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
- Direction:** for added skin protection, skin prep may be used prior to application of securement device.
- Note:** replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.
- Note:** use of an alcohol swab may help removal of securement device from the skin.

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