Teleflex®

Grip-Lok® OEM Bulk Package
Arrow® CVC & PICC Securement Device

REF 3604MCS-TA-NSB

- (EN) Catheter Fixation for ARROW® PICC/CVC
- (DE) Katheter Fixierung für ARROW® PICC/CVC
- (FR) Fixation de Cathéter pour ARROW® PICC/CVC
- (IT) Fissaggio del catetere ARROW® PICC/CVC
- (NL) Catheter Fixatie voor ARROW® PICC/CVC
- (SV) Kateter Fixering för ARROW® PICC/CVC
- (ES) La Fijación del Catéter para el ARROW® PICC/CVC
- (PT) Fixação de catéter para ARROW® PICC/CVC
- (EL) Η δέσμευση για ARROW® PICC/CVC
- (FI) Katetri Pakkomielle ARROW® PICC/CVC
- (DA) Kateter Fiksering for ARROW® PICC/CVC
- (IS) Catheter Fixation pentru ARROW® PICC/CVC
- (NO) Kateter Fiksering for ARROW® PICC/CVC
- (PL) Catheter Utrwalania dla ARROW® PICC/CVC

44U00045 Rev 00

TIDI® Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
Grip-Lok is a registered trademark of TIDI Products, LLC.

Emae distributor :
TeleflexMedical Europe
IDA Business and Technology Park,
Dublin Road, Athlone, Co. Westmeath, Ireland
Phone + 353 (0)9 06 46 08 00
www.teleflexmedical.com

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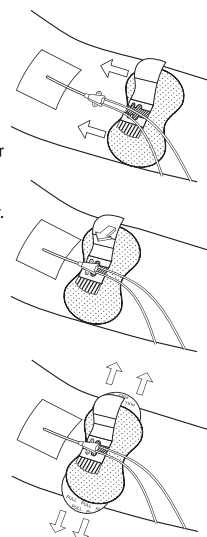
INSTRUCTIONS EN

English

1. **Select** the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
2. **Prepare** the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
3. **Open** the top flap, remove the interior liner and hold the flap open.
4. **Slide** the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
5. **Secure** the top flap over the catheter.
6. **Hold** the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

Direction: for added skin protection, skin prep may be used prior to application of securement device.

Note: replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

Note: use of an alcohol swab may help removal of securement device from the skin.

Note: Hypoallergenic

Caution: Intended for Further Processing

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