


Step 4: Prepare Drape for Attachment
 Step 5: Center on Operative Site above Sterile Field Line
 Step 6: Adhere to Patient Drape above Sterile Field Line
 Step 7: C-Armor is now ready for use!

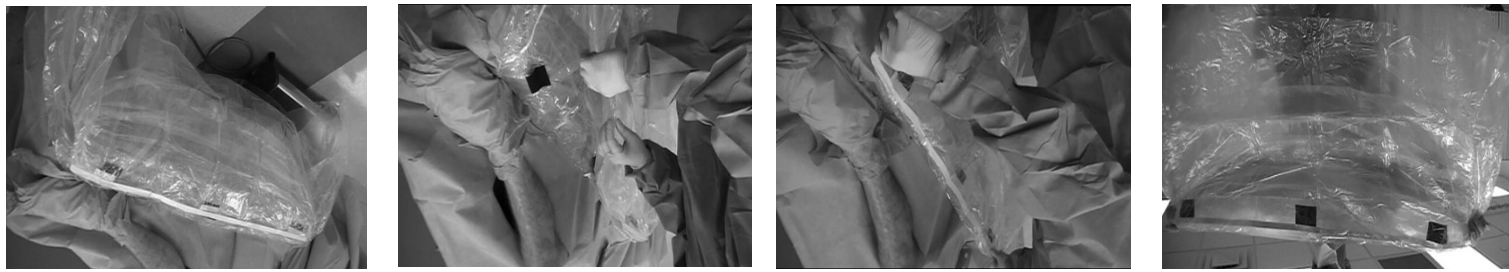

Step 1: Remove CSR Wrap
 Step 2: Unfold Drape
 Step 3: Remove Adhesive Backing



Always follow your facility's PPE protocols.

C-ARMOR®

C-ARMOR®

C-ARMOR DRAPE, non-sterile

Intended For Use:
 Fluoroscopic X-Ray Tube Cover

Manufactured By:
 TIDI Products, LLC
 570 Enterprise Drive
 Neenah, WI 54956 USA
 1.800.521.1314 +1.920.751.4300
 www.tidiproducts.com

For US and Foreign patent information, see
go.tidiproducts.com/patents

REF 5523NS
 Catalog Number

LOT 123456
 Batch Code

 2009-01-22
 Date of Manufacture

INTENDED FOR FURTHER PROCESSING

