

Disposable Operating Room Body/Knee Strap

The Posey Disposable Operating Room Body/Knee Strap is designed to assist in patient positioning for medical treatment on an examination or operating table. One dozen per package.



REF 5551 Disposable Operating Room Body/Knee Strap

Monitoring

⚠️WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Check for skin integrity and proper circulation.

Inspection

⚠️WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

Application Instructions:

1. Secure strap to side of table (Fig. 1).
2. Ensure that the printed label on the strap faces the patient.
3. Bring the "hook" part of the strap over the patient to secure closure (Fig. 2). Ensure that the strap is snug enough to properly position the patient without compromising circulation. Monitor skin and extremities for discoloration.

⚠️CAUTION The Posey Disposable Operating Room Body/Knee Strap may be easily released by the patient, and is not for use as a patient restraint.

For single patient use only; discard after use.

Disposal

⚠️WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



Strap over the operating table



Fig. 1



Fig. 2



Strap over the body



Strap over the knee