



Protect Patients. Safeguard Sterility.



**THE PATENTED, EASY-TO-USE SOLUTION
SPECIFICALLY DESIGNED TO PROTECT
STERILITY DURING PROCEDURES WITH
MULTIPLE C-ARM SWINGS.**



Help Keep the Sterile Field Sterile

Protecting patients from the risk of contamination during surgical procedures is a top priority for today's caregivers, who seek solutions that preserve and maintain sterility while easing workflow.

AORN recommendations: "Unsterile equipment...should be covered on the top, bottom, and sides with sterile barrier materials before being introduced to or brought over a sterile field." ¹

Improving the sterility of draping techniques can help reduce the risk of increases in hospital costs and post-operative infections. ²

Using C-Armor

Effective sterility maintenance

- Adhere the C-Armor drape above the sterile field line to help maintain the sterile field
- Help reduce the risk of contamination with five-sided coverage of equipment introduced to the sterile field

Ease of use


- Attach and detach the C-Armor drape simply and quickly during procedures with the fastener tabs
- Allow for multiple rotations with a single drape due to an innovative design
- Reposition the drape easily, as the C-arm rotates, with a patented inverted pouch design

Efficiency

- Help promote procedural standardization by using only one barrier per procedure
- Reduce the need for excess materials, steps and workflow changes

Form and Function

- 1 Strong adhesive ensures a secure grip on the table above the sterile field line
- 2 Drape lies flat against the table when not in use for minimal disruption of surgical access
- 3 Three fastener tabs offer easy deployment and security when the drape is not deployed
- 4 Aluminum band assists in supporting the drape above the sterile field level when deployed
- 5 Drape helps protect the C-arm from fluid damage
- 6 Five-sided protection helps maintain the sterile field

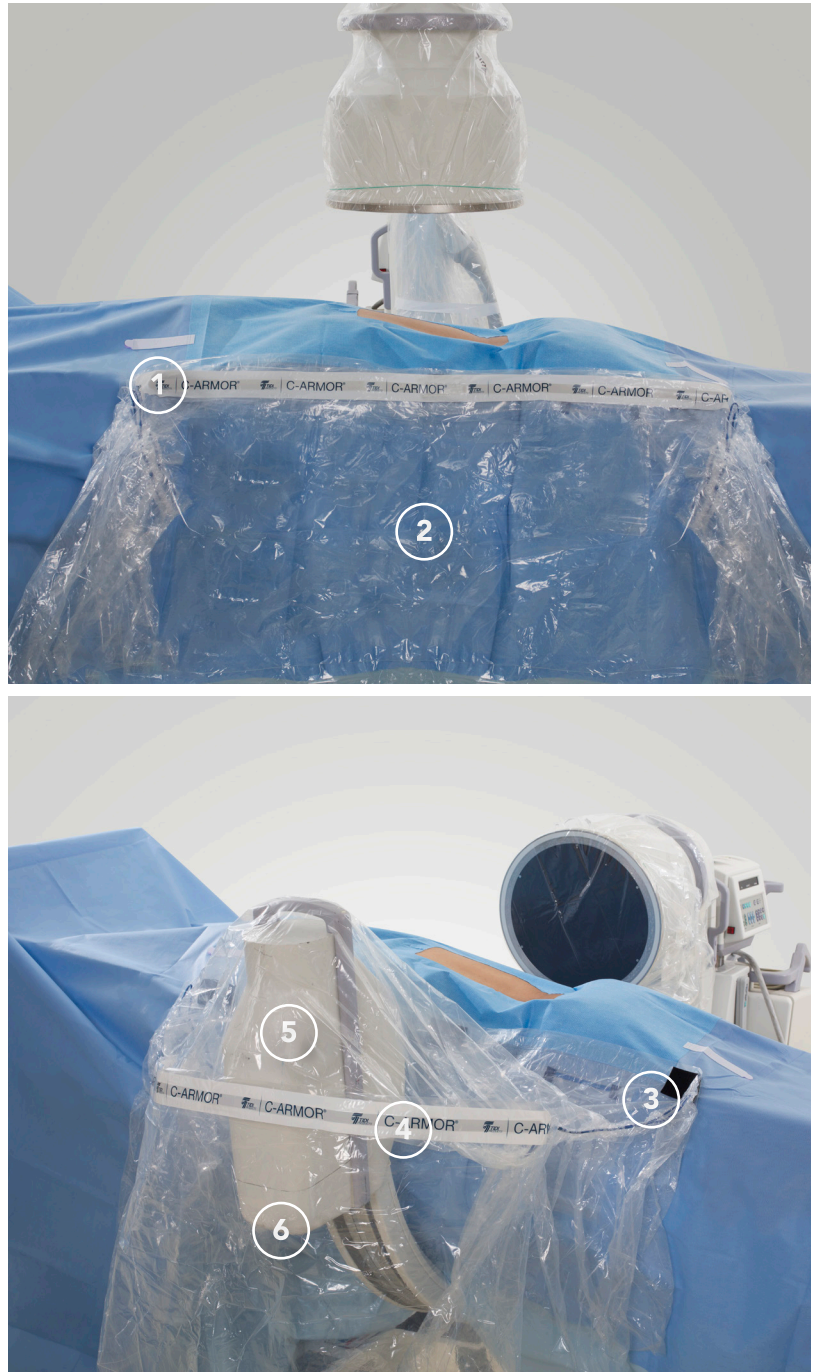

Discover how the C-Armor drape can help you protect patients by preserving sterility. Contact your sales representative or read more at tidiprotects.com. For U.S. and foreign patent information, visit tidiprotects.com/virtual-patent-marking.

SKU #	DESCRIPTION	QTY
5523	C-Armor Drape	12
5536	C-Armor Drape Kit <i>(Includes C-Arm Cover 5434: 42" x 74")</i>	12

REFERENCES

1. Guidelines for Perioperative Practice, 2019, AORN, Inc. I.Tech.IV.a.1.
2. Gershkovich, G., Hampton, D., C-Arm Fluoroscopy Draping: A Comparison of Three Techniques, Philadelphia PA