

Rapid Test

Flowflex™ SARS-CoV-2 Antigen Rapid Test (Self-Testing)

The SARS-CoV-2 Antigen Rapid Test is a lateral flow test for the qualitative detection of the nucleocapsid antigen from SARS-CoV-2 in anterior nasal swab specimens directly from individuals suspected of COVID-19 within the first seven days of the onset of symptoms. The test can also test specimens from individuals without symptoms.

- Specimen: Anterior nasal swab specimens
- Test Time: Results at 15 min.
- Shelf life: 24 months
- Storage temperature: 2-30°C
- Accuracy: 98.8%
- Sensitivity: 97.1%
- Specificity: 99.5%



CE 0123


Test Preparation

A. Open your test kit:



B. You should have:

Note: For 1T and 5T, the hole is on the kit box.

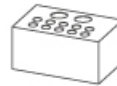


Tube Holder (for 25T only)



Disposable Swab



Test cassette

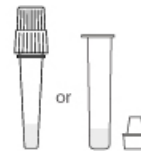

Extraction Buffer Tube



Package Insert




Waste Bag

C. Wash or sanitize your hands.



D. Read the instructions.

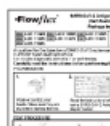

E. Check the expiration date.



F. Open the pouch.



G. Check the cassette



Specimen Collection

COLLECTION BY AN ADULT CAREGIVER



SELF COLLECTION



1



Insert the entire absorbent tip of the swab into one nostril. Using gentle rotation, push the swab less than 2.5 cm from the edge of the nostril.

2



Rotate the swab 5 times brushing against the inside of the nostril.

3



Remove the swab and insert it into the other nostril.

4




Remove swab from the nostril.



Rapid Test

Test Procedure and Interpretation

Type 1

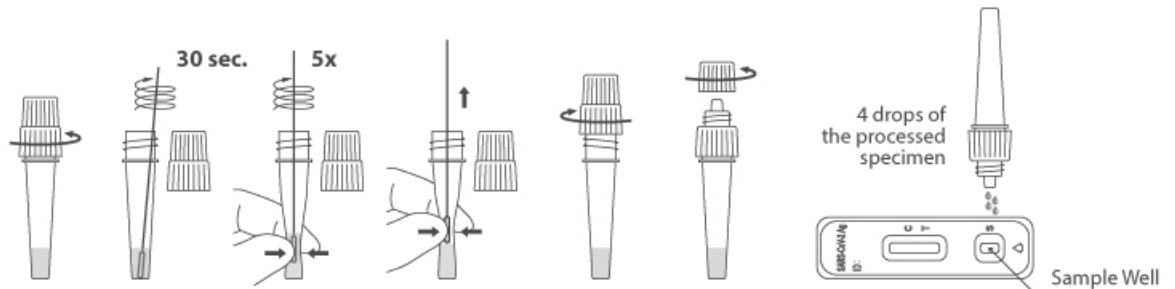

Type 2

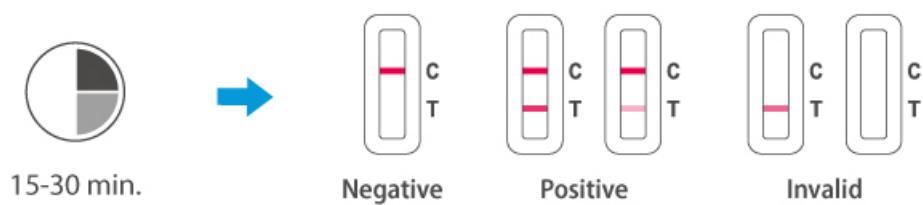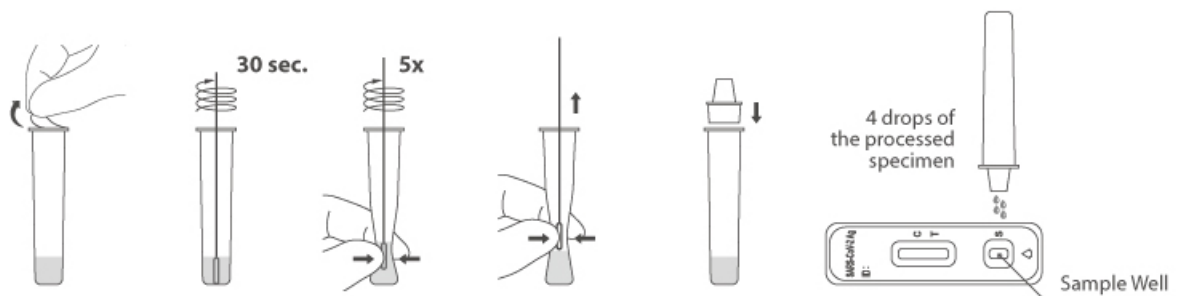

*Two type of tubes are available, which depends on the package you ordered.

Ordering Information

Product Name	Catalog No.	Format	Specimen	Package
Flowflex SARS-CoV-2 Antigen Rapid Test (Self-Testing)	L031-11855 ✓	Cassette	Anterior nasal swabs	1 Test/Kit
	L031-118A5 ✓	Cassette	Anterior nasal swabs	5 Tests/Kit
	L031-118Q5 ✓	Cassette	Anterior nasal swabs	25 Tests/Kit
	L031-118M5 ✓	Cassette	Anterior nasal swabs	1 Test/Kit
	L031-118P5 ✓	Cassette	Anterior nasal swabs	5 Tests/Kit
	L031-118R5 ✓	Cassette	Anterior nasal swabs	25 Tests/Kit

✓ CE marked